


## MESI mTABLET Print Service installation guide

MESI Print service lets the user convert any printer to a network device, which enables printing results directly from the MESI mTABLET and provides direct measurement PDF creation in a preferred folder on the computer.

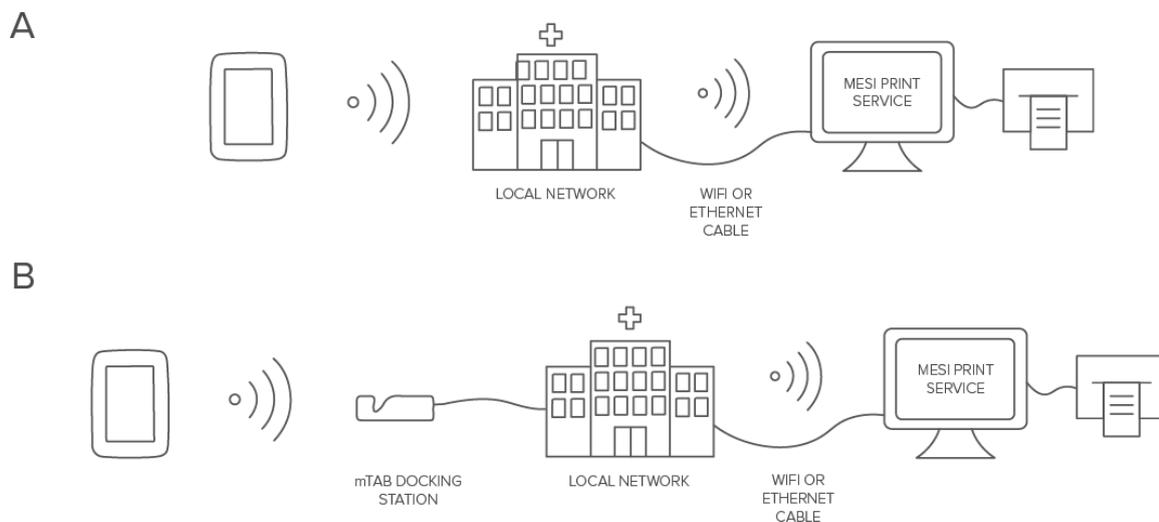


Compatible operating systems: Windows 7, Windows 10, Windows 11

## Procedure

1

Open the MESI website ([www.mesimedical.com](http://www.mesimedical.com)), click on the Menu tab and navigate to the Support section, where you select MESI mTABLET. From the list of options select Print service and click the DOWNLOAD button.

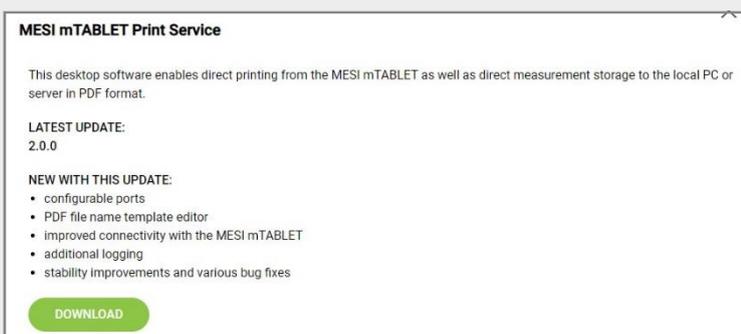


2

Click on the downloaded file and follow the installation wizard.


3

Choose your preferred language.

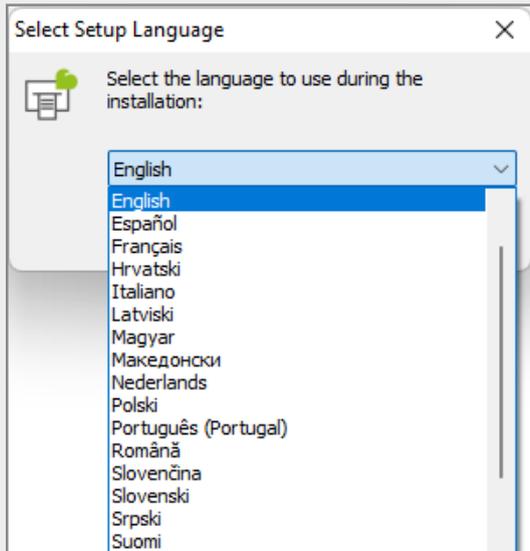


6

Check the installation overview and install the MESI mTABLET Print Service.

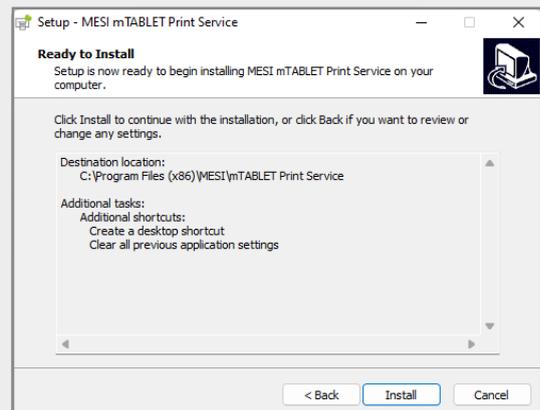


4

Choose the destination folder.

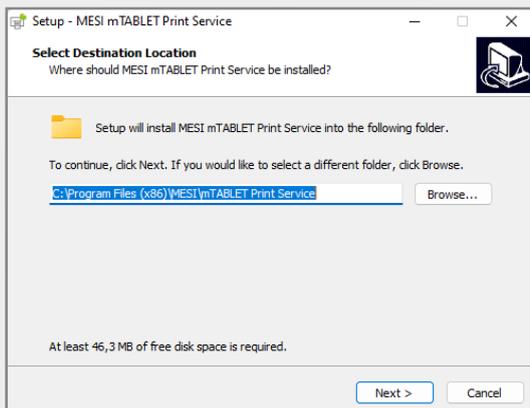


7

Launch the MESI mTABLET Print Service after installation.

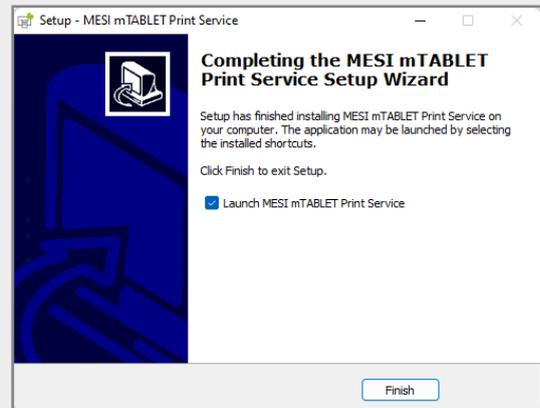


5

Toggle the options you want to select (Create a desktop shortcut, Clear all previous application settings).

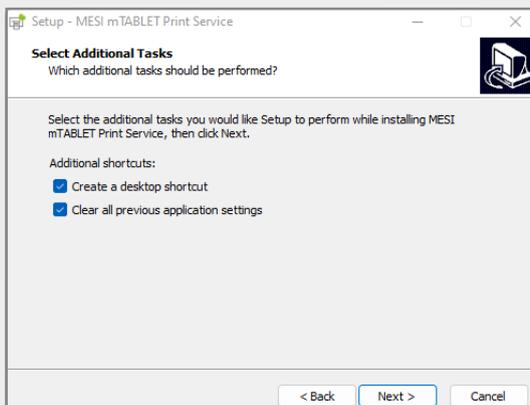

8

If the MESI mTABLET Print Service window doesn't appear, look for the  icon in the taskbar corner or taskbar corner overflow (at the bottom right of your screen). Right-click on the icon and choose Open settings... Once you launch the MESI mTABLET Print Service you can set up the parameters in the Settings window. You can check the IP address and name of the computer where MESI mTABLET Print Service is installed, choose where the reports are stored, and, in the Advanced settings section, how the reports are formatted. When your finished adjusting the settings, click the Save button for your changes to take effect.

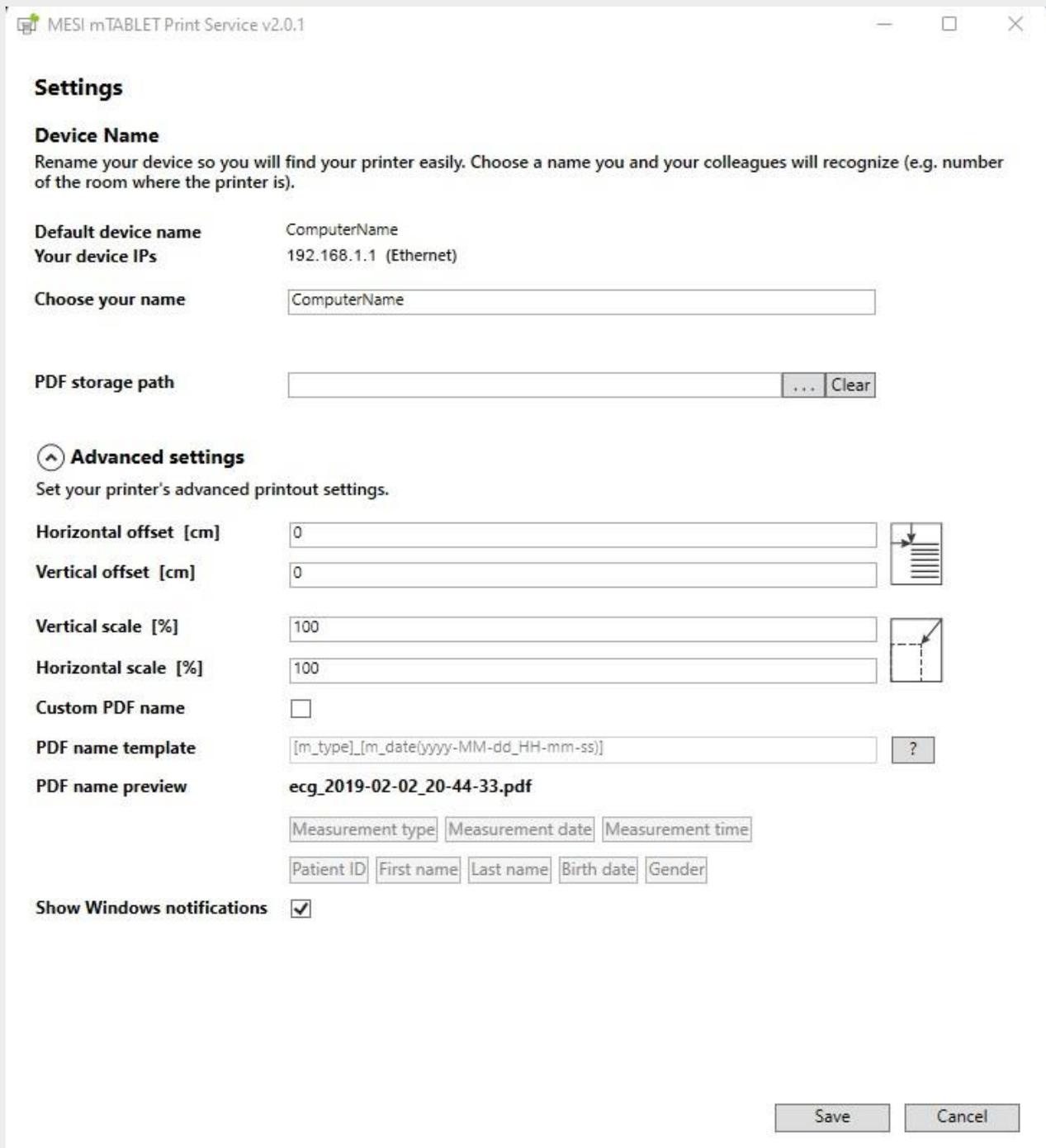

MESI mTABLET Print Service v2.0.1

## Settings

### Device Name

Rename your device so you will find your printer easily. Choose a name you and your colleagues will recognize (e.g. number of the room where the printer is).

**Default device name** ComputerName  
**Your device IPs** 192.168.1.1 (Ethernet)

**Choose your name**

**PDF storage path**  ... Clear

### Advanced settings

Set your printer's advanced printout settings.

**Horizontal offset [cm]**  

**Vertical offset [cm]**

**Vertical scale [%]**  

**Horizontal scale [%]**

**Custom PDF name**

**PDF name template**  ?

**PDF name preview** **ecg\_2019-02-02\_20-44-33.pdf**

Measurement type Measurement date Measurement time

Patient ID First name Last name Birth date Gender



**Show Windows notifications**

Save Cancel

## 9

After the installation you have to set up the printer on MESI mTABLET. Go to Device settings on MESI mTABLET, select Printers, click Find printers on the wireless network, and chose a printer from the list by tapping on the + symbol.

When selecting the printer please pay attention to the computer and printer name.

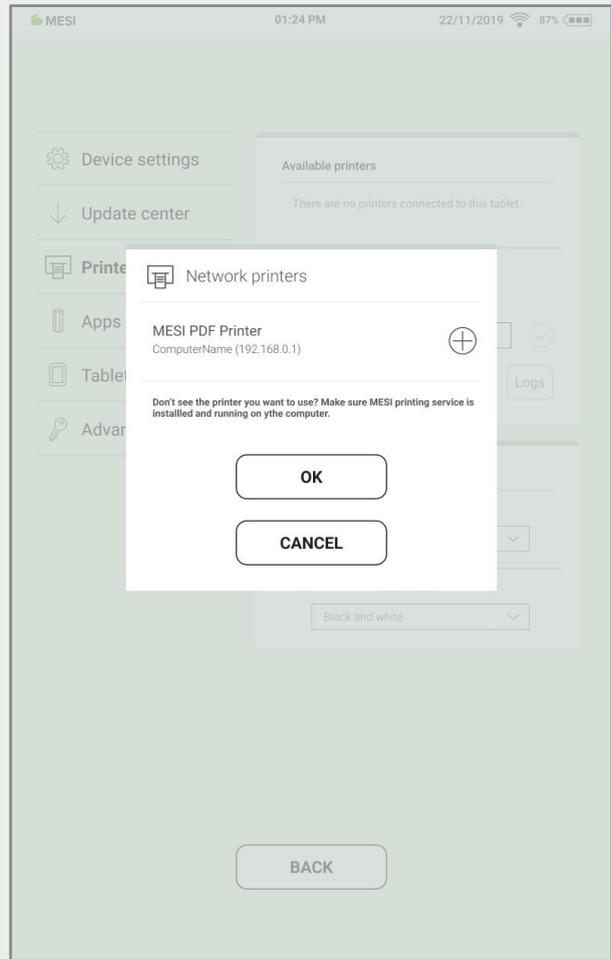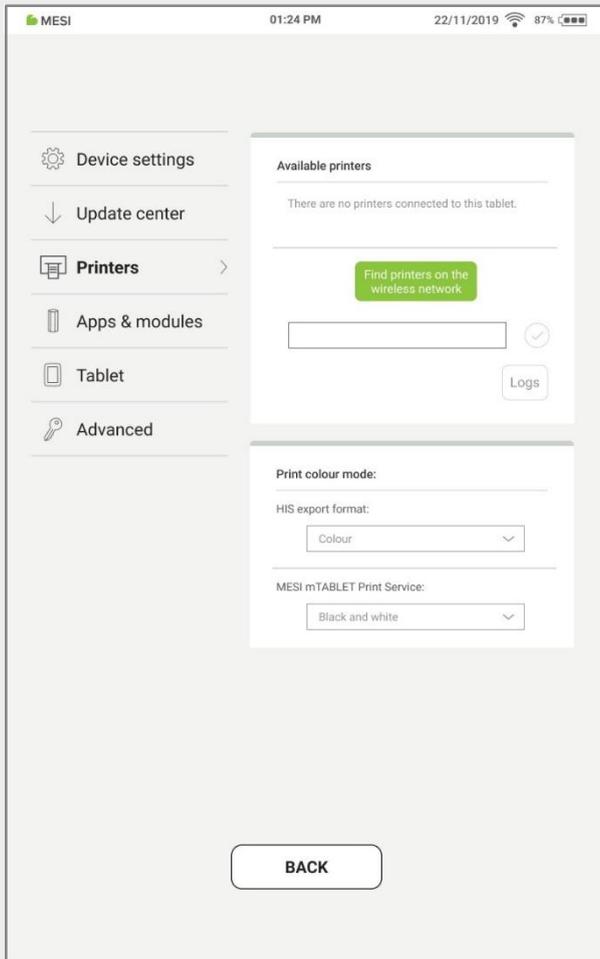

## 10

If you don't see any available printers, please refer to chapter Setting up the firewall exception on the next pages.

# Setting up the firewall exception

In order to allow MESI mTABLET Print Service through the built-in Windows Defender Firewall, follow the steps below.


1

Open Control Panel and choose Windows Defender Firewall from the list.



2

From the menu on the left choose Allow an app or feature through Windows Defender Firewall.

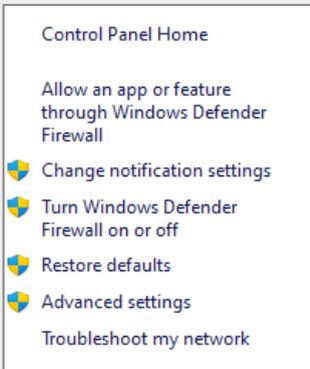

3

Click on the Change settings button, search for MESI mTABLET Print Service on the list of Allowed apps and features and toggle all network locations (Domain, Private and Public). Click OK to confirm the changes you made.

